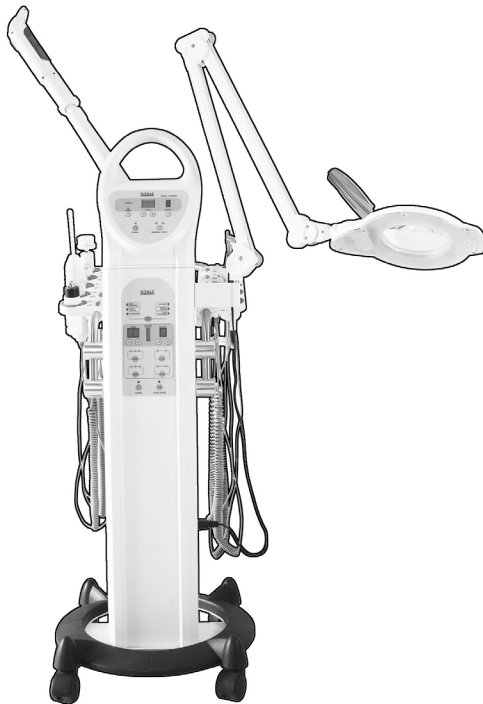
H5050

# Maia

9 in 1 Multi-functional beauty equipment / Torre multifunción 9 en 1 /  
Colonne multifonction 9 en 1 / Multifunktionssturm 9 in 1 / Torre Multifunzione 9 in 1

EN	2
ES	13
FR	25
IT	37
DE	49

DDUUEETT EUROPE would like to thank you for choosing Sonia products. We hope that this instruction manual will help you get the most out of this product.

If you have any doubts regarding its use and/or operation, please contact your distributor.

## BEFORE USE

- Please read the entire instruction manual carefully before using the product. Take note of any warnings included in this manual.
- Always keep this instruction manual handy and if you give the device to another person, hand over the manual as it is an essential part of the product.
- Some components of the product may differ from the image that appears in catalogues, websites, brochures, etc.
- In order to enhance the product, the manufacturer reserves the right to modify its features and/or components without prior notice.

## CORRECT USE OF THE DEVICE

- This product has been designed to be used by beauty professionals at beauty salons and should not be used for other purposes.
- Please note that any misuse of the device entails the loss of the guarantee.
- Do not make any modification to the device without the authorisation of DDUUEETT EUROPE SL.
- DDUUEETT EUROPE SL shall not be held liable for any material or personal damage due to the misuse of the product.

## CONTRAINDICATIONS

- This product should not be used on:
  - People with metal implants in their body.
  - People with pacemakers or heart problems.
  - Pregnant women.
  - People with sensitive skin, lesions, inflammations, or any skin problem or disease.
- With people undergoing medical treatment or a diagnostic process, the product must be used with medical authorisation.

## PRECAUTIONS

- Before using it for the first time and after each use, check the device and the mains cable for any damage.
- Unplug the device from the mains before starting to clean it or undertaking any type of maintenance.
- Do not use the product outdoors or in places

where it may be affected by humidity or inclement weather.

- If the power supply cable is defective, unplug the appliance immediately and inform your dealer or specialised technician.
- Position the power supply cable properly so that people cannot trip over it and make sure you plug it into an earthed socket.
- At the end of the working day, disconnect the device and unplug the power supply cable from the mains.
- If you detect any anomaly in the operation of the product, please contact your distributor or specialised technician.

## SAFETY INDICATIONS

- The device and accessories should be kept out of the reach of children.
- Only plug the device into a properly installed socket. The local mains voltage must correspond to the technical specifications of the device.
- The power socket must be accessible in case you have to quickly unplug the device at any given time.
- Always pull out the actual plug and not the mains cable.
- Never tamper with any electrical component or open up/repair any component of the device.
- Only your distributor or specialised technician should repair the device.
- Do not switch on the device if there is visible damage to it or to the mains cable.
- Never touch the device or the mains cable with wet hands.

## CONTENTS

- Main tower x 1 unit
- Base with wheels x 1 unit
- Facial Steamer x 1 unit
- LED magnifying lamp x 1 unit
- Power cable x 1 unit
- Face probe x 1 unit
- Eye probe x 1 unit
- Spatula x 1 unit
- Contact wristband x 1 unit
- High frequency handpiece x 1 unit
- Mushroom-shaped electrode x 1 unit
- Spoon-shaped electrode x 1 unit
- Ball-shaped electrode x 1 unit
- Straight electrode x 1 unit
- Hand-piece for galvanic probes and metal bar x 1 unit
- Roll, ball, and button galvanic probe x 1 unit

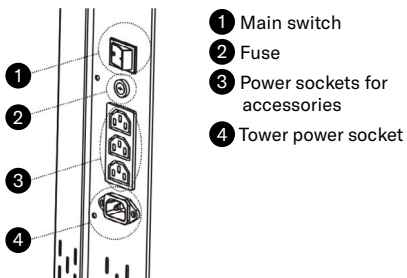
- Spray tank flexible hose x 1 unit
- Vacuum flexible hose x 1 unit
- Cotton for essential oils x 2 units
- Spray tank x 2 units
- Vacuum heads x 3 units
- Vacuum handpiece x 1 unit
- Sponge filter x 2 units
- Brush handpiece x 1 unit
- Small pumice stone brush x 1 unit
- Small brush x 1 unit
- Large pumice stone brush x 1 unit
- EVA rubber brush x 1 unit
- Large brush 1 x 1 unit
- Large brush 2 x 1 unit
- Instruction manual x 1 unit

- ① Steamer arm
- ② Magnifying lamp
- ③ Dashboard
- ④ Hand-piece holder
- ⑤ Electrical connections
- ⑥ Steamer filling lid
- ⑦ Holder for magnifying lamp
- ⑧ Steaming and suction function connections


## TECHNICAL SPECIFICATIONS

Product	9 in 1 Multi-functional beauty equipment
Reference	H5050
Voltage	AC 220-240 V~
Input frequency	50/60 Hz
Power	900 W
Dimensions	120 x 44 x 55 cm
Net weight	27,70 Kg


- Includes
- Digital Facial Steamer H1106
  - LED magnifying lamp H6001T
  - Ultrasound
  - Skin exfoliator
  - High-Frequency
  - Rotary Brush
  - Galvanic
  - Spray
  - Vacuum


- Ultrasounds

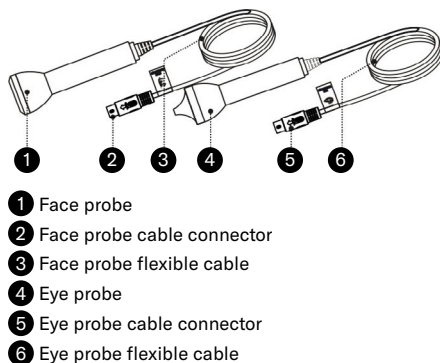
- Skin Exfoliator
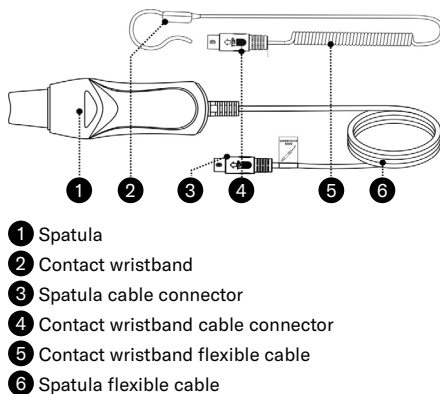
## COMPONENTS

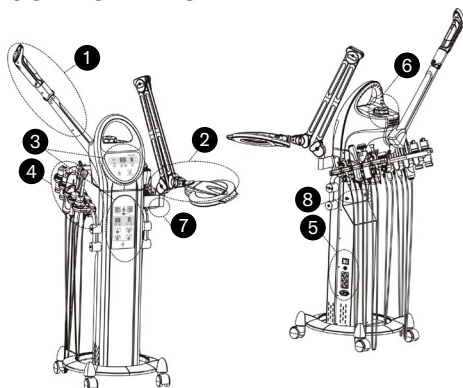
• High Frequency

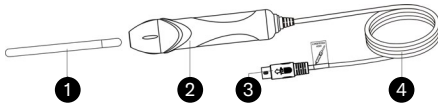- ① Electrode hose
- ② High frequency handpiece
- ③ High frequency cable connector
- ④ High frequency flexible cable

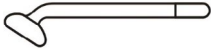- Mushroom-shaped electrode

- Spoon-shaped electrode

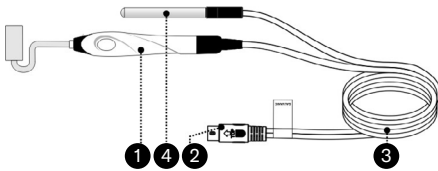
- Ball-shaped electrode

- Straight electrode


• Galvanic

- ① Handpiece with roller probe
- ② Roller probe handpiece cable connector
- ③ Roller probe flexible cable
- ④ Metal bar

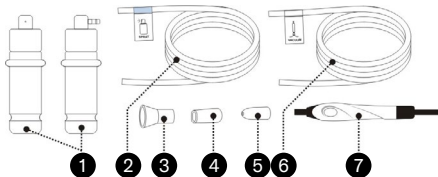
• Spray & Vacuum

- ① Spray tank
- ② Spray tank flexible hose
- ③ Large head
- ④ Medium head
- ⑤ Small head
- ⑥ Vacuum flexible hose
- ⑦


⑦ Vacuum handpiece

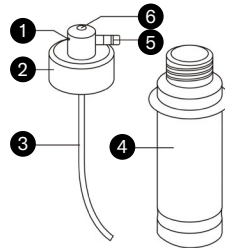Spray tank

- ① Spray output
- ② Tank lid
- ③ Vacuum hose
- ④ Spray tank
- ⑤ Spray tank connection hose
- ⑥ Air outlet

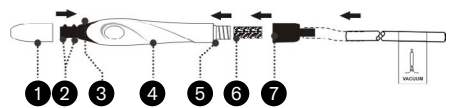
Vacuum handpiece

- ① Head
- ② Head seals
- ③ Air outlet
- ④ Vacuum handpiece
- ⑤ Joints and thread
- ⑥ Sponge filter
- ⑦ End of handpiece


• Rotary Brush

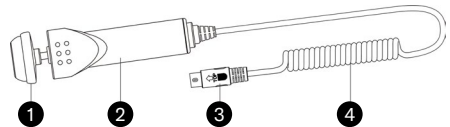Brush handpiece

- ① Head
- ② Handpiece
- ③ Brush cable connection
- ④ Brush flexible hose


## Brushes

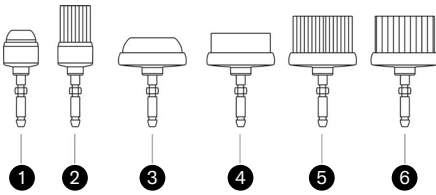
- 1 Small pumice stone brush
- 2 Small brush
- 3 Large pumice stone brush
- 4 EVA rubber brush
- 5 Large brush 1
- 6 Large brush 2

## CONNECTION AND ASSEMBLY INSTRUCTIONS

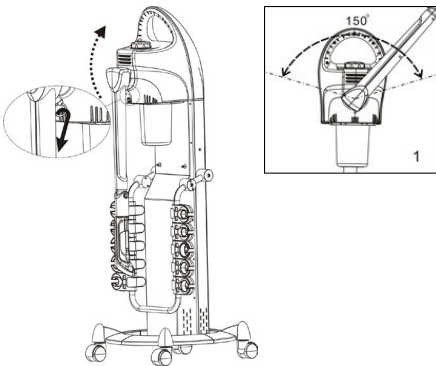
- Attach the main tower to the base with wheels using the 4 screws.
- Insert the magnifying lamp into its holder. Plug the lamp's power cord into one of the accessory power sockets on the side of the tower.
- Fit the facial steamer in the main tower.
- Unlock the steamer arm (the arm is locked as you remove the steamer from the box). Following the arrows in the image, pull out the pin to unlock the arm and turn it upwards to finish unlocking. The arm should turn freely, allowing a 150° rotation angle.

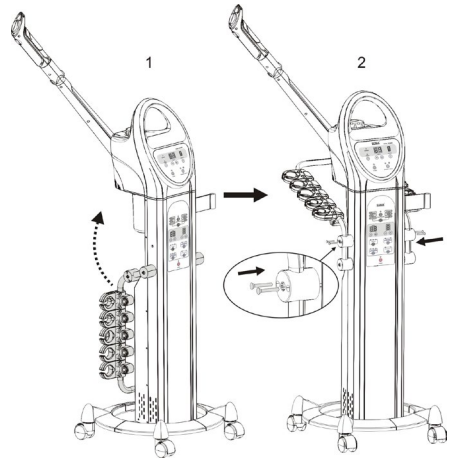- Unlock the hand-piece holder: Turn the hand-piece holder upwards until reaching the final position, as shown in the following images. Then screw the holder.

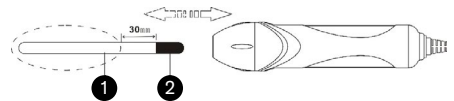
- Assembly and disassembly of the high frequency electrode

- 1 Glass part of the electrode
- 2 Metal part of the electrode

- During assembly and disassembly of the high-frequency electrode, handle the part with your hands by the area marked with dashed line, keeping a distance of 30mm from the metal part of the electrode

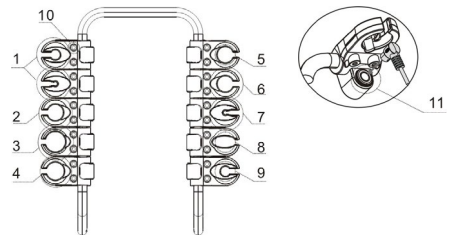
- Place hand-pieces and accessories on the hand-piece holder as shown in the image.


- 1 Roller probe + metal bar
- 2 Brush handpiece
- 3 High frequency handpiece
- 4 Spray tank

- 5 Face probe
- 6 Eye probe
- 7 Contact wristband
- 8 Spatula
- 9 Vacuum handpiece
- 10 Electrode tube
- 11 Connection of accessories

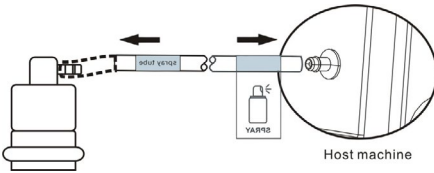
- Connect one end of the air hose with the “spray tube” label to the spray tank inlet, and the other end to the “SPRAY” tower spray connection.

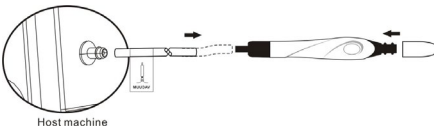- Connect one end of the air hose with the “vacuum tube» label to the suction connection of the “VACUUM” tower and the other end to the intake of the suction hand-piece. Check the end of the vacuum hand-piece to see whether the sponge filter is installed; otherwise, install it. Note the direction of the arrow on the intermediate filter. It should be pointing towards the handpiece.

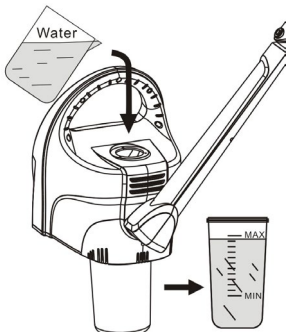- Fill the water tank by opening the steamer filling lid and pouring distilled water. The water level must stand between the “MIN” and “MAX” lines.


## FACIAL STEAMER FUNCTION

### • Precautions

- The use of distilled water is recommended to extend the useful life of the steamer components.
- Before operating the facial steamer, confirm that the water tank has been correctly screwed and tightened, and that the water level stands between the “MIN” and “MAX” level marks.
- Check the safety distance between the mouthpiece and the patient’s face, as inadequate distance may cause skin burns.
- Before adding essential oil, please ensure that the steamer is in standby mode.
- Wait until the steam is produced steadily before starting the treatment.
- The ozone function should be used according to the professional’s criteria depending on the type of skin to be treated.
- Adjust the steamer arm according to the angles that appear later in this manual. Do not force any movement.
- To remove the water tank, wait until it cools down and then unscrew the tank with your hands.
- Should there be a water leak, unplug the tower from the mains. Check if the water tank is properly screwed. If the problem persists, contact your dealer.
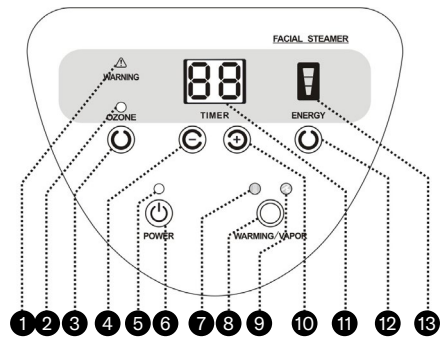
### • Control panel

- 1 Water-level warning indicator
- 2 Ozone function indicator
- 3 Ozone function on/off button
- 4 Decrease time button
- 5 Power on indicator
- 6 On / off button
- 7 Heating indicator
- 8 Heating/Steaming button
- 9 Steaming indicator
- 10 Increase time button

11 Timer

12 Steam volume adjustment button

13 Steam volume indicator

• Configuration and operation

- Plug the power cord into the tower's power socket and turn on the main switch.

- The water level inside the tank must stand between the "MIN" and "MAX" levels.

- Press  on the control panel and the power indicator lights up blue. The unit remains in standby mode and the control panel shows the following configuration:

- The timer will show 15 minutes.
- The steam volume indicator shows the maximum level.
- The ozone function indicator is off.
- The heating  and steam indicators  are off.
- Press the  button to switch between the standby mode and the heating and steaming functions.

- If the indicator  lights up red, the heating function is active. The timer will show 30 min by default and start counting down.

- Once the heating temperature has been reached, it will be maintained until the timer reaches 0.

- If the indicator  lights up green, the steaming function is active. The timer will show 15 min by default and start counting down.

- After a few minutes, steam will start to come out of the arm head nozzle.

- Set the timer using the  or  buttons to establish the operating time of the different functions.

- The heating function  allows a 1 to 90-minute range.

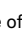
- The steaming function  allows a 1 to 30-minute range.

- This function allows defining the operating time for both functions: heating and steaming.

- Use the "ENERGY" button to adjust the steam volume to be produced by the nozzle

- If the indicator  lights up in full, the steam volume is at maximum level.

- If the indicator  lights up in half, the steam volume is at half level.

- If only one line of the indicator  lights up, the steam volume is at the minimum level.

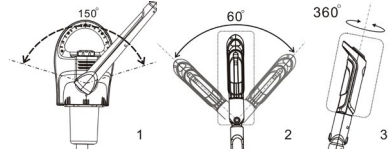
- Activate the Ozone function by pressing the "OZONE" button. The ozone function indicator will light up green. Press again to deactivate the function and the indicator will turn off.

- Adjust the steamer arm at different angles:


- 150° to the left and right

- 60° forward and backward.

- 360° adjustable head

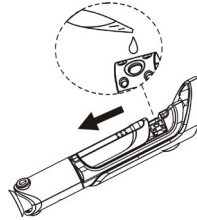- Add essential oils by opening the sliding lid at the top of the arm head nozzle and dropping one or two oil drops onto the cotton.

**Bear in mind**

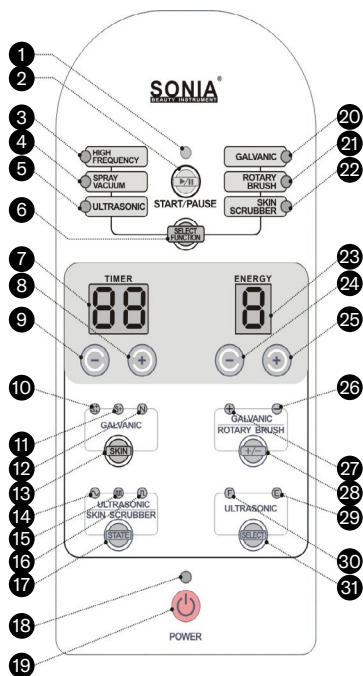
- The heating function is a heat conservation function.
- If, before activating the steaming function, the heating function has been activated, the steam is produced faster compared to a direct steaming function activation.
- If the timer shows "E2" when using the heating or steaming functions, the "WARNING" indicator flashes and an acoustic signal sounds, this indicates that the water level is too high. The active function will stop immediately.
  - Any locked functions will be unlocked as the water level returns between "MAX" and "MIN". Press  to resume operation.
- If the timer shows "E1" when using the heating or steaming functions, the "WARNING" indicator flashes and an acoustic signal sounds, this indicates that the water level is too low. The active function will stop immediately.
  - Fill with distilled water to unlock functions and resume operation.
- If you need to change the essential oil, remove the cotton by opening the sliding lid at the top of the arm head nozzle, use tweezers to remove the cotton and replace it with a new one. Add oil as required.
- Adjust the sound volume of the acoustic signals of the steamer: with the unit turned off, activate the main switch located on the side, and then press and hold the "ENERGY" button until the Timer reads "5". This is the default sound volume, which you can adjust using  or .

from 0 to 9, where 0 is the min. volume and 9 the max. volume. As you do so, the selected value flashes and is saved automatically.

## FUNCTIONS: ULTRASOUNDS, SKIN EXFOLIATOR, HIGH FREQUENCY, GALVANIC, SPRAY, VACUUM Y ROTARY BRUSH

- Before activating any function, press  on the control panel and the power indicator will light up blue. The machine enters standby mode, and you can select the desired function on the control panel.

- Control panel

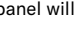
- 1 Start/Stop Indicator
- 2 Start / Stop button
- 3 High-frequency function indicator
- 4 Spraying and Suction function indicator
- 5 Ultrasound function indicator
- 6 Function selection button
- 7 Timer
- 8 Increase power button
- 9 Decrease power button
- 10 Very sensitive skin indicator

- 11 Sensitive skin indicator
- 12 Normal skin indicator
- 13 Skin type selection
- 14 Continuous wave indicator
- 15 Discontinuous wave indicator
- 16 Pulse wave indicator
- 17 Wave type selection
- 18 Indicator on
- 19 On / Off button
- 20 Galvanic current function indicator
- 21 Rotary brush function indicator
- 22 Skin exfoliator function indicator
- 23 Power display
- 24 Decrease power button
- 25 Increase power button
- 26 Anion conduction indicator (negative charge) / Anti-clockwise rotation indicator
- 27 Cation conduction indicator (positive charge) / Clockwise rotation indicator
- 28 Ion conduction selection button / Selection of brush rotation direction
- 29 Eye probe run indicator
- 30 Face probe run indicator
- 31 Ultrasound function selection

- Ultrasounds function

- Power and configuration

- In standby mode, press  to select the ultrasound function. The indicator  will light up green and the control panel will show the following configuration:
  - The timer will show 15 minutes.
  - The power indicator will show 5.
  - The face probe indicator  will light up.
  - The continuous wave indicator  will light up.
  - The run / stop indicator will light red on the push button .
- Set the timer using the  or  buttons to choose the operating time of the machine, in a range of between 1 and 30 minutes.
- Set the power using the  or  buttons to choose the power of the machine, in a range of between 1 and 9.
- Probe selection: use the  button to select the ultrasound probe to be used.
  - When the  indicator lights up, the face probe is selected.
  - When the  indicator lights up, the eye

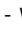
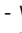
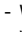
probe is selected.

- Wave type selection: use the  button to select the type of probe.
  - When the  indicator lights up, the type of wave selected is continuous.
  - When the  indicator lights up, the type of wave selected is discontinuous.
  - When the  indicator lights up, the type of wave selected is pulse.
- Use the  button to start or stop the device.
  - When in operation, the Start/Stop indicator will light up green.
  - When paused or in standby mode, the Start/Stop indicator will light up red.
- Operation
  - The unit must be in standby mode with the Start/Stop indicator lit up red.
  - Thoroughly clean the area of skin to be treated with the appropriate product according to skin type and remove any remaining product which may prevent it from penetrating again when applying the ultrasound treatment.
  - Apply conductive gel across the whole area to be treated.
  - Select the probe to be used (face or eyes), the operating time and set the power to minimum.
  - Hold the hand-piece of the ultrasonic probe with your hand and press . The Start / Stop indicator will light up green.
  - Massage the area to be treated with small circular movements for the previously selected treatment time, setting the desired power.

## Bear in mind

- Never use on dry skin. Always apply enough conductive gel for the entire treatment time.
- The treatment time should never be longer than 15 minutes and the power used should not be too high. Set the time and power according to the area of skin to be treated.
- Skin exfoliator function
  - Power and configuration
    - In standby mode, press  to select the skin scrubber function. The indicator  will light up green and on the control panel:
      - The timer will show 15 minutes.
      - The power indicator will show 1.
      - The continuous wave indicator  will light up.
      - The run / stop indicator will light red on the push button .

- Set the timer using the  or  buttons to choose the operating time of the machine, in a range of between 1 and 30 minutes.
- Set the power using the  or  buttons to choose the power of the machine, in a range of between 1 and 9.
- Wave type selection: use the  button to select the type of probe.
  - When the  indicator lights up, the type of wave selected is continuous.
  - When the  indicator lights up, the type of wave selected is discontinuous.
  - When the  indicator lights up, the type of wave selected is pulse.
- Use the  button to start or stop the device.
  - When in operation, the Start/Stop indicator will light up green.
  - When paused or in standby mode, the Start/Stop indicator will light up red.
- Operation
  - The unit must be in standby mode with the Start/Stop indicator lit up red.
  - Soften the skin in the area to be treated using a sprayer or clean water.
  - The patient should hold the wristband with their hand.
  - Select the type of wave to be used, the operating time and set the power to minimum.
  - Hold the spatula with your hand and press . The Start / Stop indicator will light up green.
  - Then gradually set the power to the level required for the treatment.

## Bear in mind

- If the Start/Stop indicator flashes, it indicates that the circuit is not closed. The patient should hold the wristband for the device to operate correctly.
- Do not use the skin scraper spatula for a long time, especially in the case of sensitive skin.
- Control the force you use with the spatula when scraping. Use at a 45-degree angle to the skin.
- Remember to keep applying product to the skin to keep it moisturised while performing a surface massage.
- High-frequency function
  - Power and configuration
    - In standby mode, press  to select the high-frequency function. The indicator  will light up green and on the control panel:
      - The timer will show 15 minutes.

- The power indicator will show 1.
- The run / stop indicator will light red on the push button 
- Set the timer using the  or  buttons to choose the operating time of the machine, in a range of between 1 and 30 minutes.
- Set the power using the  or  buttons to choose the power of the machine, in a range of between 1 and 9.
- Use the  button to start or stop the device.
  - When in operation, the Start/Stop indicator will light up green.
  - When paused or in standby mode, the Start/ Stop indicator will light up red.

#### - Operation

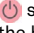
##### • Direct electrotherapy:

- Select the appropriate electrode for the treatment, set the timer and decrease power to the minimum.
- Hold the high frequency hand-piece with your hand and press  until the Start/Stop indicator lights up green.
- 14.1.3. Then bring the electrode closer to the area of the skin to be treated until it meets the skin and set the power  gradually.
- At the end of the treatment, reduce the power  and press the button  until the Start/ Stop indicator lights up red and the unit enters pause mode.
- Then you can remove the electrode from the patient's face.

##### • Indirect electrotherapy:




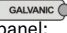
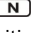
- The procedure is the same as above, except that the patient should hold the straight electrode in their hand from the beginning of the treatment.
- The operator's electrode should not come into contact with the patient's electrode.
- During the treatment, the operator should keep one hand on the patient's face.

#### **Bear in mind**

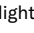- Humming sounds indicate that the high frequency electrode is working normally.
- If using direct electrotherapy, when the electrode is less than 2.5 cm away from the skin, a spark will occur. This is why you should set the power to the minimum and increase gradually. If the spark still occurs, you can use a dry cotton cloth to cover the face.
-  should be off when placing and removing the high frequency electrode.

#### • Galvanic function

##### - Power and configuration

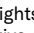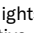- In standby mode, press  to select the galvanic current function. The indicator  will light up green and on the control panel:
  - The timer will show 15 minutes.
  - The power indicator will show 1.
  - The normal skin indicator  will light up.
  - The cation conduction (positive charge) indicator  will light up.
  - The run / stop indicator will light red on the push button 
- Set the timer using the  or  buttons to choose the operating time of the machine, in a range of between 1 and 30 minutes.
- Set the power using the  or  buttons to choose the power of the machine, in a range of between 1 and 9.
- 13.4. Skin type selection: use the  button to select the type of skin to be treated.
  - When the indicator  lights up, the selected skin type is normal.
  - When the  indicator lights up, the selected skin type is sensitive.
  - When the  indicator lights up, the selected skin type is very sensitive.
- Ion conduction selection: press  to select the polarization.
  - When the  indicator lights up, it indicates anion conduction (negative charge).
  - When the  indicator lights up, it indicates cation conduction (positive charge).
- Use the  button to start or stop the device.
  - When in operation, the Start/Stop indicator will light up green.
  - When paused or in standby mode, the Start/ Stop indicator will light up red.

##### - Operation

- The unit must be in standby mode with the Start/Stop indicator lit up red.
- Apply conductive gel or cream after cleaning the skin to be treated.
- Select the type of skin to be treated, the operating time and set the power to minimum.
- In order to correctly select the polarisation of the ion conduction with the  button, the pH value of the previously applied product should be considered, selecting one or the other polarity depending on whether it is an alkaline or an acidic product.
- The patient must hold the metal bar with his/ her hand.

- Hold the hand-piece of the galvanic probe with your hand and press . The Start / Stop indicator will light up green
- Start to slowly move the roller probe and gradually adjust power until reaching the treatment level, always maintaining a slight stinging sensation.

### Bear in mind

- Roller, ball and button galvanic probes are interchangeable between them. To change a probe, place the unit in standby mode, unscrew the existing probe and screw the desired one.
- When the polarity is set for positive electrodes, there will be a stinging sensation given off by the roller probe that is stronger than the one given off by the negative electrode. Select it according to your needs.
- The selection of ion conduction to be used should coincide with the pH of the product applied.
  - When the probe acts as a positive charge, the conductive solution must be acidic (+).
  - When the probe acts as a negative charge, the conductive solution must be basic (-).
  - If the pH is not indicated in the solution to be used, use a pH indicator strip to determine it.
- If the Start/Stop indicator flashes, it means that the circuit is not closed and there is no ion conduction. Verify that the patient is holding the metal bar.

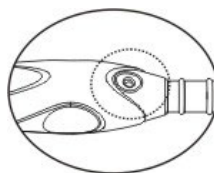
### • Spray and vacuum function

- Power and configuration
  - In standby mode, press  to select the spraying and suction function. The indicator [SPRAY VACUUM] will light up green and on the control panel:
    - The timer will show 15 minutes.
    - The power indicator will show 9
    - The run / stop indicator will light red on the push button 
    - Set the timer using the  or  buttons to choose the operating time of the machine, in a range of between 1 and 30 minutes.
    - Use the  button to start or stop the device.
    - 
    - When in operation, the Start/Stop indicator will light up green.
    - When paused or in standby mode, the Start/ Stop indicator will light up red.

### - Operation

- The unit must be in standby mode with the Start/Stop indicator lit up red.

- Select the operating time.
- Spray function:
  - Hold the spray tank in your hand and press . The Start/Stop indicator lights up green.
  - Lightly press the air outlet on the top of the tank with your fingers to start spraying through the nozzle.
- Vacuum function:
  - Firmly hold the vacuum handpiece in your hands, place it in the desired position and press . The Start / Stop indicator will light up green
  - The nozzle of the head should meet the skin.
  - Lightly press the air outlet of the handpiece with your fingers to make the vacuum of the skin in contact with the head.

### Bear in mind

- The device should be put in pause mode when changing the vacuum tank or head.
- Prepare two spray tanks, for different liquids, and you can alternate the tank according to your needs.
- Make sure that the sponge filter is installed on the end of the handpiece before starting vacuum.
- Be careful when treating areas such as the mouth, nose and eyes. The patient should keep their eyes and mouth closed.
- The vacuum power should be kept under control, as a high power could cause skin lesions.
- Test first on your own hand, and then start with the patient once the vacuum power has been set.
- Select different heads for the different areas of the skin:
  - Small head: for small areas of the skin 
  - Medium head: for narrow areas of the skin 
  - Large head: for large areas of the skin 
- Regularly replace the sponge filter with a new one.

- Rotary brush function

- Power and configuration

- In standby mode, press  to select the rotating brush function. The indicator  will light up green and on the control panel:
  - The timer will show 15 minutes.
  - The power indicator will show 5
  - The clockwise rotation indicator  will light up.
  - The run / stop indicator will light red on the push button 
- Set the timer using the  or  buttons to choose the operating time of the machine, in a range of between 1 and 30 minutes.
- Set the power using the  or  buttons to choose the power of the machine, in a range of between 1 and 9.
- Use the  button to start or stop the device.
  - When in operation, the Start/Stop indicator will light up green.
  - When paused or in standby mode, the Start/Stop indicator will light up red.

- Operation

- The unit must be in standby mode with the Start/Stop indicator lit up red.
- Select the direction of rotation, the operating time and the power.
- Hold the brush handpiece in your hand and press . The Start /Stop indicator lights up and the brush begins to spin.

### Bear in mind

- Before starting treatment, check that the brush has been inserted correctly.
- If during operation of the device a change of direction of rotation is selected, the device will enter safety mode and after stopping rotation will begin to rotate in the new direction.
- Slow down to the minimum speed before touching the patient's skin.
- Gradually increase the speed until you reach the desired level.

## RECYCLING

If you need to dispose of the product or any of its parts or accessories, please check and respect the laws and regulations applicable to waste management and environmental protection.

## CLEANING AND MAINTENANCE

- Before cleaning the device, unplug it from the mains.
- Clean the device with a soft, dry cloth. Do not use liquids to clean it.
- After each use, the various components and accessories must be sterilised and left to dry.
- For maintenance or repairs of any part of the product, please contact your distributor or specialised technician.

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